




Source of information: leaflet (note: the instructions might differ according to the brand of the acquired inhaler)

Using the inhaler

1

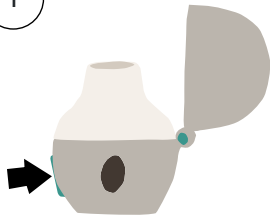


After removing your inhaler device from the pouch: Open the dust cap (lid) by pressing the green piercing button.

2




Open the mouthpiece by pulling the mouthpiece ridge up and away from the base so the center chamber is showing.

3

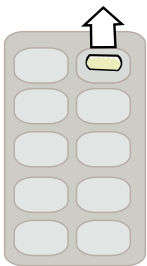


Insert the capsule into your inhaler device: each day, separate only 1 of the blisters from the blister card by tearing along the perforated line. Remove the capsule from the blister: do not cut the foil or use sharp instruments to take out the capsule from the blister. Bend 1 of the blister corners with an arrow and separate the aluminum foil layers. Peel back the printed foil until you see the whole capsule. If you have opened more than 1 blister to the air, the extra capsule should not be used and should be thrown away. Place the capsule in the center chamber of your inhaler device.

4

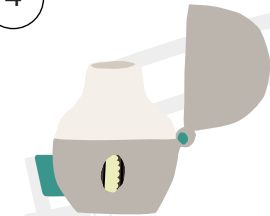


Close the mouthpiece firmly against the gray base until you hear a click. Leave the dust cap (lid) open.

5

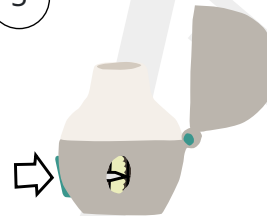

Hold your device with the mouthpiece pointed up. Press the green piercing button once until it is flat (flush) against the base.

6

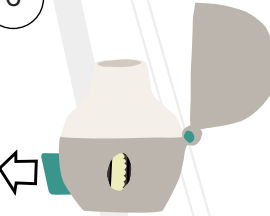

Release the button. This is how you make holes in the capsule so that you get your medicine when you breathe in. Do not press the green button more than one time. Do not shake your inhaler. The piercing of the capsule may produce small gelatin pieces. Some of these small pieces may pass through the screen of your device into your mouth or throat when you breathe in your medicine. This is normal. The small pieces of gelatin should not harm you.

7



Breathe out completely in 1 breath, emptying your lungs of any air. Important: Do not breathe into your device.

8



Close your lips tightly around the mouthpiece.



9



Breathe in deeply until your lungs are full. You should hear or feel the capsule vibrate (rattle).

10



Hold your breath for a few seconds and, at the same time, take your device out of your mouth.

11



Breathe normally again.

12


If you didn't hear any vibrating sound from the capsule oscillating in the chamber, you should review the steps in the leaflet that comes with the device and/or seek for help with a health professional.

13




Open the inhaler to see if any powder is still in the capsule. If any powder remains in the capsule, you should review the steps in the leaflet that comes with the device and/or seek for help with a health professional.

14

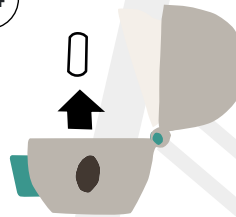

After use, open the inhaler, remove and discard the empty capsule. Do not leave a used capsule in the chamber.

15

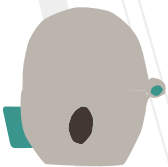

Then, close the mouthpiece and dustcap for storage.